

CALCIUM STABLE LIQUID STANDARD

Kódszám/Catalog No.: 52001
Kiszerezés/Size: 1x5 ml

Gyártási szám: 7395 #####

Lejáratidő: ÉÉÉÉ-HH

Felhasználás:

Kalcium reagens kalibrálására szolgáló stabil folyékony kalcium standard.

Felhasználásra kész.

A standard koncentrációja LOT specifikus.

Koncentráció:

Reagens	Érték	Egység
Kalcium stabil folyékony reagens (Arsenazo III)	###	mmol/l
Kalcium stabil folyékony reagens (OCPC/AMP)	###	mmol/l

Visszavezethetőség:

AAS

Stabilitás:


Felbontás nélkül 2-8°C-on tárolva a lejáratidőig.


Felbontás után 2-8°C-on tárolva minimum 4 hétig.


Figyelmeztetések:

Csak in vitro diagnosztikai célokra használható.

Ne használja a standardet a lejáratidő eltelte után!

A címkén a következő szimbólumok találhatóak:
 In vitro diagnosztikai használatra

 Lejáratidő (a hónap utolsó napja)

 Tárolási hőmérséklet

 Gyártási szám

 Kódszám

Lot No: 7395 #####

Expiry date: YYYY-MM

Preparation:

Calcium standard for the calibration of calcium reagent.

Ready to use.

Concentration is LOT specific.

Concentration:

Reagent	Value	Unit
Calcium stable liquid reagent (Arsenazo III)	##	mg/dl
Calcium stable liquid reagent (OCPC/AMP)	##	mg/dl

Traceability:

AAS

Stability:

Without opening at 2-8°C until the expiry date.


After first opening at least 4 weeks at 2-8°C.

Warnings:


For in vitro diagnostic use only.

Do not use after the expiry date!

The following symbols are used on labels:
 For in vitro diagnostic use

 Expiry date (last day of the month)

 Temperature limitation

 Batch Code

 Code