

Gyártási szám: 7175 #####

Lejárati idő: ÉÉÉÉ-HH

Felhasználás:

Kreatinin reagens kalibrálására szolgáló stabil folyékony kreatinin standard.

Felhasználásra kész.

A standard koncentrációja LOT specifikus.

Koncentráció:

Reagens	Érték	Egység
Kreatinin stabil folyékony reagens (Jaffe)	##	µmol/l
Kreatinin stabil folyékony reagens (enzimatis)	##	µmol/l

Visszavezethetőség:

GC-IDMS

Stabilitás:




Felbontás nélkül 2-8°C-on tárolva a lejáráti időig.

Felbontás után 2-8°C-on tárolva minimum 4 hétig.

Figyelmeztetések:

Csak in vitro diagnosztikai célokra használható.

Ne használja a standardet a lejáráti dátum eltelte után!

A címkén a következő szimbólumok találhatóak: In vitro diagnosztikai használatra Lejárati idő (a hónap utolsó napja) Tárolási hőmérséklet Gyártási szám Kódszám

Lot No: 7175 #####

Expiry date: YYYY-MM

Preparation:

Creatinine standard for the calibration of creatinine reagent.

Ready to use.

Concentration is LOT specific.

Concentration:

Reagent	Value	Unit
Creatinine stable liquid reagent (Jaffe without compensation)	###	mg/dl
Creatinine stable liquid reagent (enzymatic/PAP)	###	mg/dl

Traceability:

GC-IDMS

Stability:


Without opening at 2-8°C until the expiry date.

After first opening at least 4 weeks at 2-8°C.

Warnings:

For in vitro diagnostic use only.

Do not use after the expiry date!

The following symbols are used on labels: For in vitro diagnostic use Expiry date (last day of the month) Temperature limitation Batch Code Code